

Surgical Face Mask
Type IIR (BFE \geq 98%)



Surgical Face Mask Type IIR (BFE \geq 98%)



Technical information

Typology: Medical Face Mask

Size: 175 x 94 mm

Standard: EN 14683-2019 + AC:2019. Type IIR

Bacterial filtration efficiency: \geq 98%


Features:

Type IIR: maximum surgical quality
Splash resistant

Structure: mask body, nose clip and mask belt

Validity period: 24 months

Allergy warning: Non-woven textile

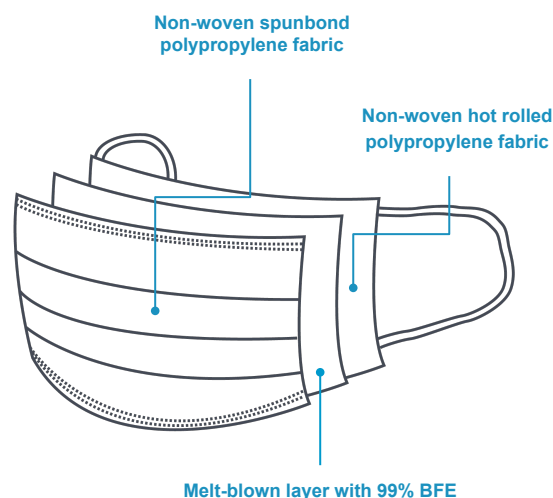

Fitting instructions

1. With the blue side facing outwards, place it over the nose and mouth, with the nose clip facing upwards. Then place the loops behind the ears.
2. Gently press on the nose clip in order to adjust it to the nose.
3. Pull the lower part of the mask so that it is beneath the chin and check that it fits well.



Important information



- Hypo-allergenic waterproof polypropylene material, including flexible nasal clip.
- CE Test Report issued by TÜV SÜD.
- Conditions for storage: To be stored in an area with relatively low humidity, free from corrosive gases and well ventilated, avoiding high temperatures ($>40^{\circ}\text{C}$).



Mascarilla quirúrgica
Tipo IIR (BFE \geq 98%)



Surgical Face Mask Type IIR (BFE \geq 98%)

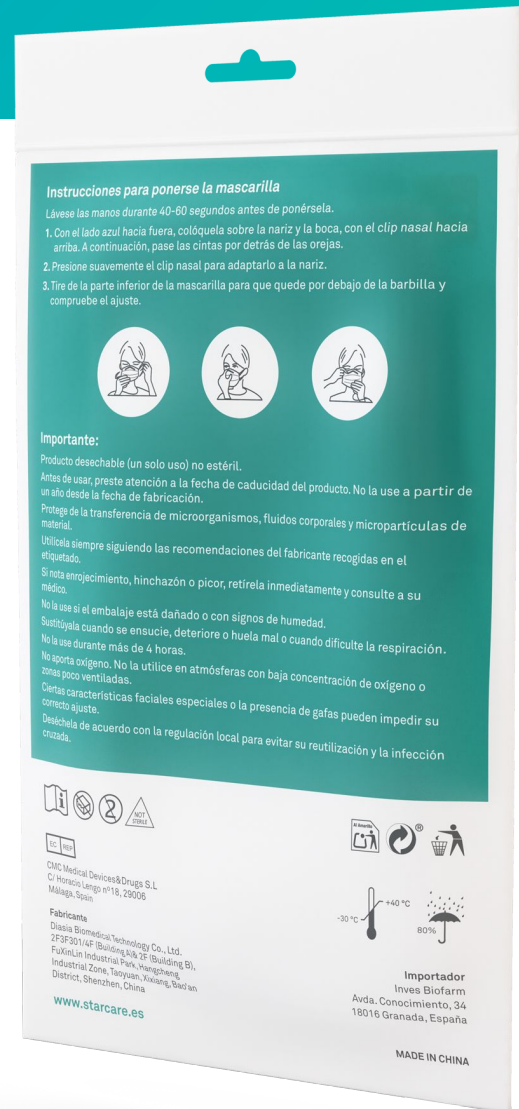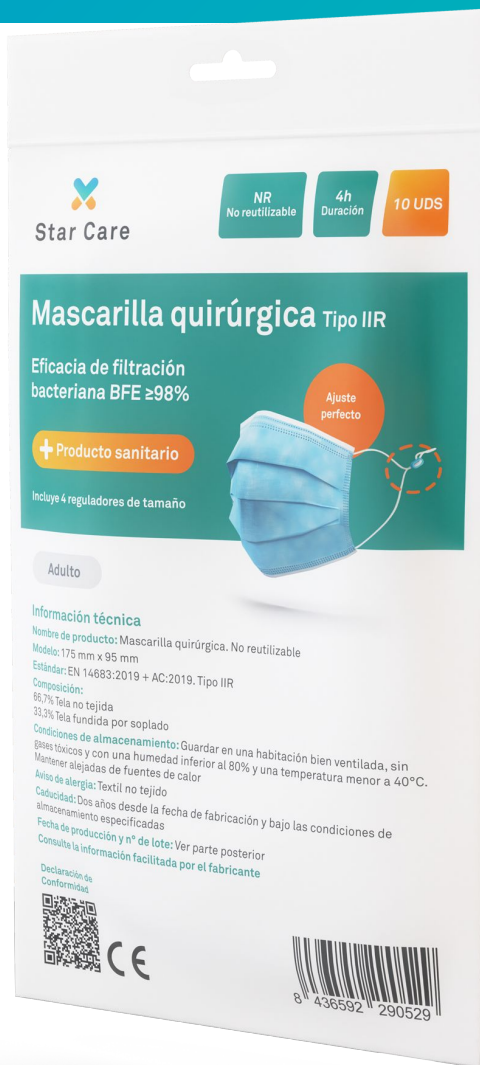

**Surgical Face Mask
Type IIR (BFE ≥ 98%)**



**Surgical Face Mask
Type IIR (BFE ≥ 98%)**



CHARACTERISTICS	STANDARD	RESULTS
Bacterial Filtration Efficiency (BFE)	EN 14683:2019+AC:2019 (E) Annexe B	BFE ≥ 99%
Differential pressure (ΔP)	EN 14683:2019+AC:2019 (E) Annexe C	ΔP < 38 Pa/cm ²
Microbial cleaning	EN 14683:2019+AC:2019 (E) Annexe D	CFU/g < 20
Synthetic blood penetration	ISO 22609:2004	Not detected

Source: Test Report No.: 200193327 (TÜV SÜD)

10 pcs bag specifications

Quantity: 10 units/bag
Dimensions: 210 x 120 x 23 mm
Weight: 25 g

Master carton specifications

Quantity: 100 bags/master carton
Total quantity: 1000 units
Dimensions: 525 x 435 x 270 mm

SKU: MASK-ST-00032.0010

